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continues, IMMEDIATELY REMOVE THE LENS AND CONTACT YOUR EYE CARE PROFESSIONAL.

During therapeutic use of ACUVUE® OASYS Brand Contact Lenses with HYDRACLEAR® PLUS, an adverse effect may be due to the original disease or injury may be due to the effects of wearing a contact lens. There is a possibility that the existing disease or condition might become worse when a soft contact lens for therapeutic use is used to treat an already diseased injured eye. To avoid serious eye damage, you should contact your Eye care Professional IMMEDIATELY if there is an increase in symptoms while wearing the lenses.

DIRECTIONS FOR USE

When you first get your lenses, be sure that you are able to put the lenses on and remove them (or have someone else available who can remove the lenses for you) before leaving your Eye Care Professional's office.

DO NOT use if the sterile blister package is opened, damaged or after the expiry date.

Step 1: Getting Started

It is essential that you learn and use good hygiene in the care and handling of your new lenses.

Cleanliness is the first and most important aspect of proper contact lens care. In particular, your hands should be clean, dry, and free of any soaps, lotions, or creams before you handle your lenses.

Before you start:

- Always** wash your hands thoroughly with warm water, a mild soap, rinse carefully and dry with a clean lint-free towel before touching your lenses to reduce the chance of getting an infection.
- You should avoid the use of any soaps containing cold cream, lotion, or cosmetics before handling your lenses. These substances may come into contact with the lenses and interfere with successful wearing. It is best to put on your lenses before putting on makeup.

Step 2: Opening the Packaging

Always confirm the lens's parameters (e.g., diameter (DIA), base curve (BC), lens power (D), etc.) printed on the multi-pack and on the individual lens package match your prescription. DO NOT use if there is a mismatch.

Multi-pack

Each multi-pack contains individually packaged lenses. Each lens comes in its own lens package designed specifically to keep it sterile while sealed.

Always store individual unopened ACUVUE® OASYS with Transitions™ blisters out of direct sunlight.

Lens Package

To open an individual lens package, follow these simple steps:

- Shake the lens package and check to see that the lens is floating in the solution.
- Carefully peel back the foil closure to reveal the lens.
- Place a finger on the lens and slide the lens up the side of the bowl of the lens package until it is free of the contain surface of the foil when opened, or to the plastic package itself. This will not affect the sterility of the lens. It is still perfectly safe to use. Carefully remove and inspect the lens following the handling instructions.

Lens Handling Tips

- Handle your lenses with your fingertips, and be careful to avoid contact with fingernails. It is helpful to keep your fingernails short and smooth.
- Develop the habit of always working with the same lens first to avoid mix-ups.
- After you have removed the lens from the packaging, examine it to be sure that it is a single, moist, clean lens that is free of any nicks or tears. If the lens appears damaged, DO NOT use it.

Step 3: Placing the Lens on the Eye

Remember, always start with the same eye.

Once you have opened the lens case or lens package, removed, and examined the lens, follow these steps to apply the lens onto your eye:

1. BE SURE THE LENS IS NOT INSIDE-OUT by following one of the following procedures:

- Place the lens on the tip of your index finger and check its profile. The lens should assume a natural, curved, bowl-like shape. If the lens edges tend to point outward, the lens is inside out.
- Gently squeeze the lens between the thumb and forefinger. The edges should turn inward. If the lens is inside out, the edges will turn slightly outward.

- Place the lens on the tip of your index finger and, looking up at the lens, locate the numbers 1-2-3. 1-2-3 indicates correct orientation, while a reverse of 1-2-3 indicates the lens is inside out. If the lens is inside out (reverse 1-2-3), invert the lens and locate the numbers again to confirm correct lens orientation.

Note that the 1-2-3 marking is not present on all ACUVUE® lenses.

With the lens on your index finger, use your other hand to hold your upper eyelid so you will not blink.

Look up at the ceiling and gently place the lens on the white of the lower part of your eye.

Slowly release your eyelid and close your eye for a moment.

Blink several times to centre the lens.

Use the same technique when applying the lens for your other eye. There are other methods of lens placement. If the above method is difficult for you, ask your Eye Care Professional for an alternate method.

Step 4: Checking Your Lenses

After you have successfully applied your lenses, you should ask yourself:

- Do I see well?
- How do the lenses feel on my eyes?
- How do my eyes look?

If after placement of the lens, your vision is blurred, check for the following:

- The lens is not centred on the eye (see "Step 5: Centring the Lens", next in this leaflet).
- If the lens is centred, remove the lens (see "Removing Your Lenses") and check for the following:
 - Cosmetics or oils on the lens. Dispose of the lens and apply a new fresh lens.

- The lens is on the wrong eye.
- The lens is inside out (it would also not be as comfortable as normal). See "Step 3: Placing the Lens on the Eye".

If you find that your vision is still blurred after checking the above possibilities, remove the lens and consult your Eye Care Professional. Note: If a lens is noticeably uncomfortable upon application or becomes less comfortable while you are wearing them, it was first applied, remove the lens immediately and contact your Eye Care Professional. If your examination of your eyes and the lenses shows any other problems, IMMEDIATELY REMOVE YOUR LENSES AND CONTACT YOUR EYE CARE PROFESSIONAL.

Step 5: Centring the Lens

A lens, which is on the cornea (centre of your eye), will very rarely move onto the white part of the eye during wear. This, however, can occur if application and removal of the lenses are not performed properly. To centre a lens, follow either of these procedures:

- Close your eyelids and gently massage the lens into place through the closed lids.
- or

Gently move the off-centred lens onto the cornea (centre of your eye) while the eye is opened using a clean finger pressure on the edge of the upper lid or lower lid.

Hazardous Conditions

- If you use aerosol (spray) products, such as hair spray, while wearing lenses, keep your eyes closed until the spray has settled.
- Avoid** all harmful or irritating vapours and fumes while wearing lenses.
- Never** rinse your lenses in water from the tap.

Water Activity

Do not expose your contact lenses to water while you are wearing them.

Lubricating/Rewetting Solutions

- Your Eye Care Professional may recommend a lubricating/rewetting solution for your use. These solutions can be used to wet (lubricate) your lenses while you are wearing them.

- Do not use saliva or anything other than the recommended solutions for lubricating or rewetting your lenses. Do not put lenses in your mouth.

- Different solutions cannot always be used together and not all solutions are safe for use with all lenses. Use only recommended solutions.

- Do not change solution without consulting with the Eye Care Professional.

- Never use solutions recommended for conventional hard contact lenses only.

- Always use fresh, unexpired lens care solutions and lenses and always follow directions in the package inserts for the use of contact lens solutions.

- Sterile unpreserved solutions, when used, should be discarded after the time specified in the directions.

- Always keep the lenses completely immersed in the recommended storage solution when the lenses are being worn (stored). Prolonged periods of drying (e.g. exposing the lens to air for 30 minutes or more) will reduce the ability of the lens surface to return to a wettable state. If the lens surface does become dried out, discard the lens and use a new one.

Sharing Lenses

Never allow anyone else to wear your lenses. Sharing lenses greatly increases the chance of eye infections.

Adhering to the Prescribed Wearing & Replacement Schedules

- Never** wear your lenses beyond the amount of time recommended by your Eye Care Professional.

- Always dispose** worn lenses as prescribed by your Eye Care Professional.

- Any unused product or waste material should be disposed of in accordance with local requirements.

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Johnson & Johnson Vision Care Ireland UC, The National Technology Park, Limerick, Ireland.

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AMO Ireland, Block B, Liffey Valley Office Campus, Quarryvale, Co. **EC REP**
Dublin, Ireland
www.acuvue.com

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Any previously diagnosed condition that makes contact lens wear uncomfortable

- Reduced corneal sensitivity (corneal hyposthesia)
- Any systemic disease that may affect the eye or may be made worse by wearing contact lenses
- Allergic reactions on the surface of the eye or surrounding tissues that may be induced or made worse by wearing contact lenses or use of contact lens solutions
- Any active corneal infection (bacterial, fungal, protozoal or viral) if eyes become red or irritated
- Irritation of the eye caused by allergic reactions to ingredients in contact lens solutions (i.e., rewetting drops). These solutions may contain chemicals or preservatives (such as mercury, Thimerosal, etc.) to which some people may develop an allergic response.

For Therapeutic Use, your Eye Care Professional may prescribe your contact lenses to aid in the healing process of certain ocular conditions that may include those listed above.

3. WARNINGS - What You Should Know About Contact Lens Wear:

EYE PROBLEMS, INCLUDING CORNEAL ULCERS (ulcerative keratitis), CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION. IF YOU EXPERIENCE ANY OF THE FOLLOWING SYMPTOMS:

- Excessive discomfort
- Loss of vision
- Excessive Tearing
- Eye Redness
- Vision Changes

YOU SHOULD IMMEDIATELY REMOVE THE LENSES, AND PROMPTLY CONTACT YOUR Eye Care Professional.

- Lenses prescribed for daily wear (i.e.: your Eye Care Professional instructs you to remove your lenses at the end of each day), should not be worn while sleeping. Clinical studies have shown the risk of serious eye problems (i.e.:ulcerative keratitis) is increased when lenses are worn overnight.

- Studies have shown that contact lens wearers who smoke have a higher rate of eye problems (i.e.:ulcerative keratitis) than nonsmokers.

- Problems with contact lenses or lens care products could result in serious injury to the eye. You should be cautioned that proper use and care of contact lenses and lens care products, including lens cases, are essential for the safe use of these products.

- The overall risk of serious eye problems (i.e.: ulcerative keratitis) may be reduced by carefully following directions for lens care.

- Do not** expose contact lenses to **water**, including while wearing them. If lenses have been exposed to water, you should discard and replace them with a new pair.

- UV ABSORBING CONTACT LENSES** are not substitutes for protective eyewear. UV absorbing eyewear such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. You should continue to use UV absorbing eyewear as directed.

- Do not reuse or "top-off" old solution left in your lens case since solution reuse reduces effective lens disinfection and could lead to severe infection, vision loss, or blindness. "Topping-Off" is the addition of fresh solution to solution that has been sitting in your case.

- Discard any remaining solution after the recommended time period indicated on the bottle of multi-purpose solution used for disinfecting and soaking your contact lenses. The discard date refers to the time you can safely use the contact lens care product after the bottle has been opened. It is not the same as the expiration date, which is the last date that the product is still effective before it is opened.

- Using multi-purpose solution beyond the discard date could result in contamination of the solution and can lead to severe infection, vision loss, or blindness. To avoid contamination, DO NOT touch the tip of the container to any surface. Replace cap after using. DO NOT transfer to other bottles or containers.

- Do not store your lenses or reuse your lens case with water or any non-sterile solution. Only use fresh multi-purpose solution so you do not contaminate your lenses or lens case. Use of non-sterile solution can lead to severe infection, vision loss, or blindness.

- It is recommended that you see your Eye Care Professional routinely as directed.

² New England Journal of Medicine, September 21, 1989; 321 (12), pp. 773-783

4.PRECAUTIONS

- DO NOT** use if the sterile blister package is opened, damaged or after the expiry date.

- When you first get your lenses, be sure that you are able to put the lenses on and remove them (or have someone else available who can remove the lenses for you) before leaving your Eye Care Professional's office.

- NEVER** use tweezers or other tools to remove your lenses from the lens container.

- Remember, always start with the same eye.

- Always be sure the lens is in your eye and you see clearly before commencing your removal technique.

General Precautions:

- Due to the small number of patients enrolled in clinical investigation of lenses, all refractive powers, design configurations, or lens parameters available in the lens material are not evaluated in significant numbers. Consequently, when selecting an appropriate lens design and parameters, the Eye Care Professional should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen permeability, wettability, central and peripheral thickness, and optic zone diameter.

- The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive correction; therefore, the continuing ocular health of the patient and lens performance on the eye should be carefully monitored by the prescribing Eye Care Professional.

- If you wear your contact lenses to correct presbyopia you may not be able to get the best corrected visual acuity for either far or near vision. Visual needs are different for different people, so your Eye Care Professional should work with you when selecting the most appropriate type of lens for you.

- Eye Care Professionals should instruct the patient to remove lenses immediately if the eyes become red or irritated.

- Always** contact your Eye Care Professional before using any medicine in your eyes.

- Be aware** that certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers, and those for motion sickness may cause dryness of the eye, increased lens awareness (feeling of the lens in the eye), or blurred vision. Always inform your Eye Care Professional if you experience any problems with your lenses while taking such medications.

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©Johnson & Johnson Vision Care Companies